

Canceling a Data Transfer from the Siemens Vida (SyngoMR XA)

To Cancel a Data Transfer via the Network or to Media

1) Open the **Jobs View** window, by clicking on the Briefcase or monitor icons in the upper right-hand / lower left corners of the Data Management window



Figure 1: Data Management Window

- 2) In the **Job View** tool (Figure 2), select the appropriate jobs tab: **Media** for transfers to an external hard disk or USB drive; **Network** for transfers to a PACS server - The tool will display currently active transfers.
- 3) Select the transfer you wish to halt.
- 4) Click **Cancel** to halt the transfer; its **Status** will then change to **Failed**.
- 5) Click **Delete** to remove the transfer from the queue.



Figure 2: Job View Tool